

INSTRUCTIONS FOR USE - Power System Attachments and Accessories, Cutting Accessories

CAUTIONS AND WARNINGS

Power System Attachments and Accessories

- The attachments and accessories are intended to be used only by licensed medical professionals who are familiar with their use and application. It is the user's responsibility to become familiar with the proper use of them. Only use with the indicated power system, see applicable instructions for use.
- Charging Bays, Power Units and Power Modules refer to applicable power system instructions for use.
- Inspect handpiece, attachments and accessories for proper operation and cleanliness before use.
 - Ensure handpiece is in "SAFE" mode prior and during installation of attachment and cutting accessory.
 - Ensure proper insertion and tightening of attachment and cutting accessory into handpiece, before activation.
 - Use the required attachments, guards and protect patient tissue at all times.
- Observe attachment and cutting accessories during use to ensure proper operation.
 - Never use a bent or damage attachment or cutting accessory.
 - Contact with metal fixtures, retractors or parts may cause damage to attachment or cutting accessory and may necessitate its replacement.
- Eye protection for operating room personnel is recommended during use.
- Do not modify handpiece, attachments or accessories

Burs, K-Wires, Steinman Pins and Twist Drills

- Do not use cutting accessories at speeds exceeding the instrument manufacturer's specifications.
- Avoid application of excessive lateral force.
- Use proper bur guard with bur per the instrument manufacturer's specifications.



Sawblades

- Do not use excessive force, twist or bend blades.
- Insert blade into alignment guide or cutting fixture prior to activating instrument.
- Irrigation of blades is recommended during use and is required when using an alignment guide or cutting fixture.

K-Wire Guide

- K-Wire Guide must be used only with MicroLine Wiredriver KM-M11 Series to ensure proper wire alignment.
- Do not steam sterilize K-Wire Guides

Sterilization Parameters

Non-Sterile Product

(Wrapped or unwrapped):

<u>Sterilization type</u>	<u>Temperature</u>	<u>Exposure Time</u>
Pre-Vac	270°F (132°C)	4 minutes
250°F Gravity	250°F (121°C)	60 minutes Total
270°F Gravity	270°F (132°C)	35 minutes Total

All cycles require at a minimum an 8 minute dry time.

- Observe AAMI and hospital guidelines for steam sterilization and the sterilizer manufacturer's written instructions for cycle parameters and load configurations.
- Dispose of products per guidelines in a safe and proper manner.
- Remove Wire Guide from the K-Wire and the instrument prior to sterilization.

